

# University of Utah

## Cutaneous Nerve Evaluation

*Please read through all information BEFORE scheduling patient's biopsy*

### Right Now...

- 1) Open the materials and FREEZE:
  - Cold pack for return shipment
- 2) Store bag containing Fixative tubes at Room Temperature
- 3) Keep cardboard box, FedEx info and Styrofoam cooler for return shipment

### On Biopsy Day...

- 1) Biopsy patient in clinic
- 2) Place the biopsy in the Zamboni's Fixative (yellow liquid).
- 3) Send to lab in shipper with cold pack via FedEx, Standard Overnight delivery to:

### Use pre-printed return shipping label provided

University of Utah, School of Medicine  
Cutaneous Nerve Lab – C/O Gordon Smith, MD  
30 North 1900 East, Room 3R228  
Salt Lake City, UT 84132 USA  
1-801-585-2461

Reports will be sent to the referring physician requesting the test.

If you need immediate assistance, please call Peter Hauer directly at 410-917-3972  
Or email [biopsy@hsc.utah.edu](mailto:biopsy@hsc.utah.edu)

