

Urine Electrophoresis Testing

April 2024

To ensure adequate specimen is received for urine electrophoresis testing, please review the following specimen guidelines.

NOTE FOR 24H URINES: Please add the total volume electronically in your ordering system and physically write it on the specimens.

Immunotyping, Urine, 24 Hour or Random

Test Code: IEP-F

Minimum Volume: 15mL

Preferred Specimen: Please aliquot urine from 24h urine jug Aliquot urine into one (1) Full 10mL tube and one (1) 6mL no additive vacutainer tube (Urine Chemistry tube) and note the total volume.



Electrophoresis, Protein Urine 24 Hour

Test Code: UPEN

Minimum Volume: 15mL

Preferred Specimen: Please aliquot urine from 24h urine jug Aliquot urine into one (1) Full 10mL tube and one (1) 6mL no additive vacutainer tube (Urine Chemistry tube) and note the total volume.



Electrophoresis, Protein Urine Random

Test Code: UPNR1

Minimum Volume: 15mL

Preferred Specimen: Aliquot urine into one (1) FULL 10mL urine tube and one (1) 6mL no additive vacutainer tube (Urine Chemistry tube) and note the total volume.



Electrophoresis, Protein Urine Random, w/ Reflex Creatinine

Test Code: UPNR

Minimum Volume: 15mL

Preferred Specimen: Aliquot urine into one (1) FULL 10mL urine tube and one (1) 6mL no additive vacutainer tube (Urine Chemistry tube) and note the total volume.

