

Job Aid: Urine Collections for Infants and Small Children

Applying a Pediatric Collection Device (U-Bag)

- Wash hands thoroughly.
- Apply gloves.
- Prep site with one No-Sting Skin-Prep pad.
- Apply U-Bag to baby, sealing all areas to keep urine in bag.
- Newborn – based on size of baby
- Pediatric – larger size

Removing a U-Bag

When pediatric patient comes with urine collection device attached:

- Wash hands thoroughly.
- Apply gloves.
- Remove the collection device from patient without contaminating the specimen.
- Tear blue tab off of bottom of U-Bag to retrieve urine.
- Without further contamination, pour or decant the specimen into the proper collection container.