

Date

Patient Name

MRN

Physician Requesting:

REQUEST: Evaluation of platelets on known platelet clumper.

Instructions:

Cancer Center Staff: Place only the:

blue and/or **purple** (Please circle tubes submitted)

top tubes to be used for platelet testing in a bag with this sheet.

Specimen Processing Staff: Receive bloods and take the purple and/or blue top tubes directly into Hematology along with this sheet. (DO NOT SPIN BLUE TOP)

Hematology Staff: Run the blue and/or the purple top tubes for platelet count. Free text both counts under test code PLC. Note the tube type with each result.

Example: 79-;purple top-;85 blue top

Please see Sharon Hall or Sam Sponaugle at 74230 with questions.