


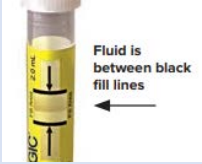



COLLECTION DEVICE	EPIC ORDER NUMBER	SOURCE	PERFORMING LABORATORY
<p><b>Hologic Aptima Unisex Swab</b></p>  <p>Lawson Item#: 103060</p>	<p>Neisseria gonorrhoea – <b>LAB123301</b>  Chlamydia trachomatis – <b>LAB123300</b>  N. gonorrhoea/C. trachomatis – <b>LAB123303</b></p> <p>Mycoplasma genitalium - <b>LAB1231403</b></p>	<p>Endocervical swab,  Male urethral swab</p> <p>Endocervical swab,  Male urethral swab</p>	<p><b>WVUH – Molecular Diagnostics Laboratory</b></p>
<p><b>Hologic Aptima Multi Test Swab</b></p>  <p>Lawson Item#: 122711</p>	<p>Vaginitis/Vaginosis</p> <ul style="list-style-type: none"> <li>• BV only – <b>LAB1231400</b></li> <li>• Candida and Trichomonas only – <b>LAB1231401</b></li> <li>• BV, Candida and Trichomonas Panel – <b>LAB1231402</b></li> </ul> <p>Neisseria gonorrhoea – <b>LAB123301</b>  Chlamydia trachomatis – <b>LAB123300</b>  N. gonorrhoea/C. trachomatis – <b>LAB123303</b></p> <p>Mycoplasma genitalium - <b>LAB1231403</b></p>	<p>Vaginal swab</p> <p>Throat/pharyngeal swab, rectal swab, or vaginal swab</p> <p>Vaginal swab</p>	<p><b>WVUH – Molecular Diagnostics Laboratory</b></p>
<p><b>Hologic Aptima Urine Collection Device</b></p>  <p>Lawson Item#: 103061</p>	<p>Neisseria gonorrhoea – <b>LAB123301</b>  Chlamydia trachomatis – <b>LAB123300</b>  N. gonorrhoea/C. trachomatis – <b>LAB123303</b></p> <p>Mycoplasma genitalium - <b>LAB1231403</b></p>	<p>Female/male first-catch urine <sup>■</sup></p> <p>Female/male first-catch urine <sup>■</sup></p> <p><sup>■</sup>Note: Using disposable pipette, transfer from urine cup to indicated fill line on transport device (image).</p>	<p><b>WVUH – Molecular Diagnostics Laboratory</b></p> 
<p><b>Hologic Aptima Multi Test Swab</b></p>  <p>Lawson Item#: 122711</p>	<p>Mycoplasma hominis - <b>LAB15254</b>  Ureaplasma species - <b>LAB15255</b>  Mycoplasma/Ureaplasma - <b>LAB18187</b></p>	<p>Vaginal swab, Male urethral swab <sup>*</sup></p> <p><sup>*</sup>Note: If ordering with above tests, an additional swab is required.</p>	<p><b>Quest Diagnostics</b></p>